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
incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

PACKAGE CONTENTS

- These components are subject to change without prior notice.
- Appearance of product may differ from the image.

| No. | PRODUCT | REF. | QTY. |
|-----|---------------------|-------|------|
| 1 | SKIN SCRUBBER PROBE | 10018 | 1 |
| 2 | SAFENER | - | 1 |
| 3 | BRACELET | 10019 | 1 |
| 4 | MAIN UNIT | - | 1 |
| 5 | FEEDER CABLE | - | 1 |

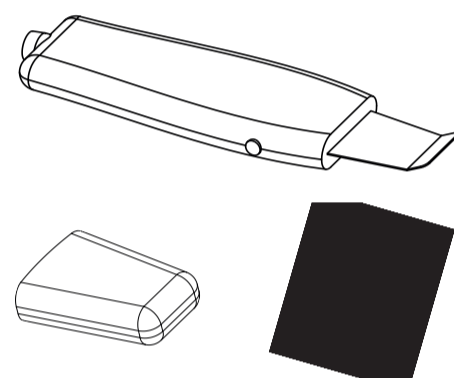WARNING

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.
- It must be placed on a beauty trolley.
- It must be used in an area where the probe wire can be accessed (within 0.5 m).
- Do not use it with 2m of accessible gas.
- Power wire and machine frame shall be used with 1,5m.

EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2006/95/EC
2. Conditions set out in the Electromagnetic Compatibility Directive: 2004/108/EC

RECYCLING

WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to

TECHNICAL SHEET

| | | |
|----------------------------|---------------------|-------------|
| Weights and sizes | Packing size (cm) | 39x36x18,5 |
| | Gross weight (kg) | 4,3 |
| | Product size (cm) | 33x17/24x13 |
| Working characteristics | Working power | 6 W |
| | Working frequency | 25 - 27 KHz |
| | Preset working time | 15 min |
| Electrical characteristics | Rated voltage | 220V ~ 240V |
| | Frequency | 50Hz/60Hz |
| | Rated power | 29 W |

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