CONTENIDO DEL PAQUETE

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
PACKAGE CONTENTS

- These components are subject to change without prior notice.

- Appearance of product may differ from the image.

No.	PRODUCT	REF.	QTY
1	FIVE LEG BASE		1
2	WHEEL LEG		5
3	STRAIGHT POST		1
4	SLIDING BAR		1
5	SET SCREW		1
6	PRESSING PLATE		1
7	FLAT GASKET		1
8	GROWER'S SCREW		1
9	SCREW		1
10	WRENCH		1
11	F-300A		1
12	WATER TANK	10060	1
13	SEAL	10060-1	1
14	FILTER MESH	10031	1
15	HERBAL DEPOT		1
16	WOOL FELT		5

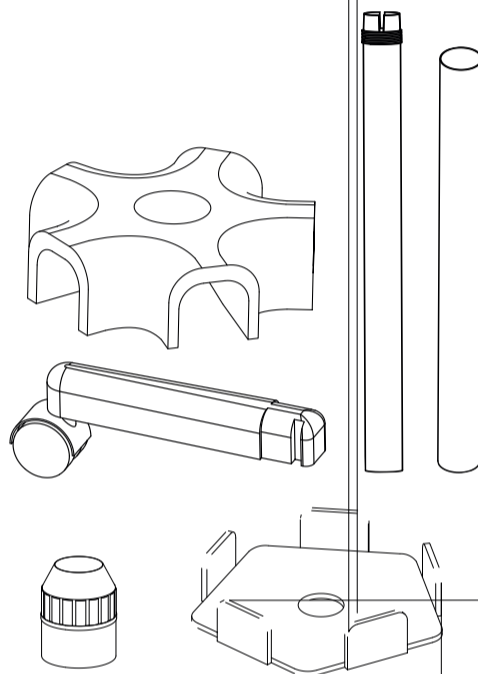IMPORTANT NOTICE

- Before using the steamer with a customer, it is very important to perform two entire cycles. Do not use it with a patient!

- It is advisable to wash the new cup vessel thoroughly before its first use, with clean water only.

WARNING

- Do not use any substance other than distilled water. Take care when using essential oils or herbs, so as not to contaminate the water used for steaming.

- NEVER point the vapor jet directly in the face of a customer.

- NEVER use the steamer unit in humid or poorly ventilated environments.

- NEVER use the steamer outdoors.

- NEVER attempt to open the steamer casing/housing by yourself.

- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

- Do not let the steamer run dry. If it does, the warranty is voided.

EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2006/95/EC

2. Conditions set out in the Electromagnetic Compatibility Directive: 2004/108/EC

RECYCLING

WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

TECHNICAL SHEET

Adjustments	Upper height limit - arm at 45° (cm)	168
	Lower height limit - arm at 45° (cm)	110
	Rotation of arm	180°
	Rotation of the nozzle	3D
	Extension of the arm (cm)	-
Weights and sizes	Packing size (cm)	87,5x34x35
	Gross weight (kg)	9
Water tank	Material	Glass
	Capacity	0,7 l
Working characteristics	Preheat time	6 min. (approx.)
Electrical characteristics	Rated voltage	220V ~ 240V
	Frequency	50Hz/60Hz
	Rated power	800W
Fragrances		Yes
Ozone		Yes
Deposit for herbs		Yes

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